


Technovit-2-Bond



The fast, 2-part PU adhesive for hoof treatment!

KERBL

Technovit-2-Bond is a quick-drying, two-part adhesive for treating hooves. Just a small amount of adhesive is all that's needed to glue wood or rubber blocks to healthy hooves in order to take the strain off diseased or damaged ones and therefore accelerate healing.

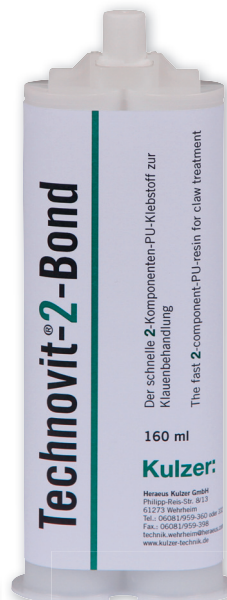

The handy starter set of Technovit-2-Bond includes accessories for around 10 applications (depending on the size of the hoof and the conditions of use):

Handy starter set :

- 1 x 160 ml cartridge containing 2-part adhesive
- 10 x wooden blocks
- 10 x mixing tips



Here's how simple it is to use:


- remove the seal on the cartridge and connect up the mixing tip.
- eject a small amount of adhesive. This must be done so that the two components mix as effectively as possible.
- now apply a bead of adhesive onto the block.
- place the block quickly onto the pre-prepared, clean and dry, healthy hoof.
- after around 30 seconds, the adhesive will have set, and after around 2-3 minutes, the hoof can weight-bear again.

The benefits at a glance:

- dries within 30 seconds
- bonds powerfully and permanently to wood and rubber blocks
- easy to use - no mixing, no waste
- odourless and safe to use, even in confined spaces
- long storage life and unaffected by temperature when stored

16242



ref. no.	description		€
16240	Technovit-2-Bond starter set for 10 applications	1	
16241	Technovit-2-Bond cartridge, 160 ml	1/10	
16242	Dispensing gun	1	
16243	Mixing tips, 10 pieces	1=10	