SURE sense®

microchip
reader

User Manual

EN: User Manual	1 - 6
DE: Bedienungsanleitung	7 - 12
FR: Guide d'utilisation	13 - 18
NL: Handleiding	19 - 24
ES: Manual de usuario	25 - 30
IT: Manuale d'uso	31 - 36
PT: Manual do usuário	37 - 42
NO: Brukerhåndbok	43 - 48
SV: Användarmanual	49- 54
DK: Brugervejledning	55-60
FI: Käyttöopas	61-66

Welcome to your new Microchip Reader

The SureSense Microchip Reader has a lightweight, ergonomic design and one-button operation that makes it easy for anyone to use. The reader's compact design makes it easy to hold in one hand whilst scanning, and is less intimidating to the animal.

The microchip reader is compatible with ISO FDX-B (15-digit) microchips and will also display the Microchip Temperature of Bio-Thermo and other compatible temperature sensing microchips.

These microchips have built-in temperature sensors and offer a non-invasive method of monitoring an animal's temperature. For more information, please visit www.sureflap.com/bio-thermo and see the temperature sensing microchip leaflet provided.

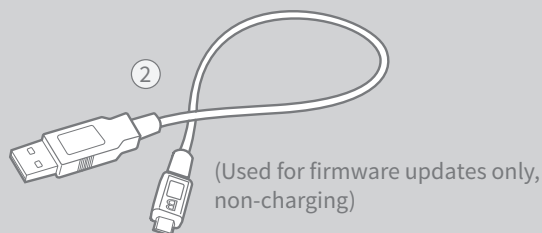
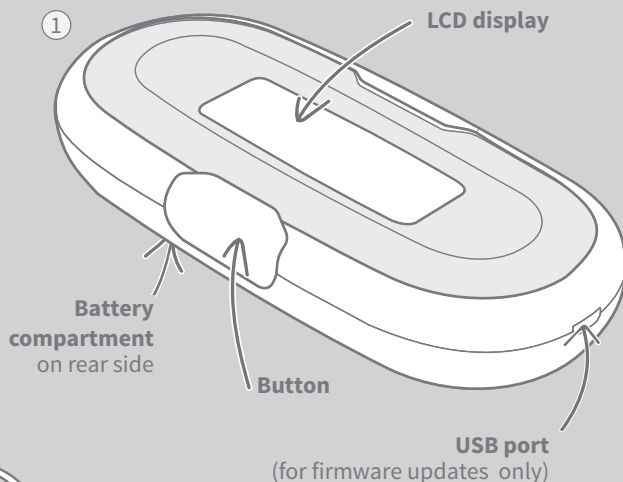
Product tour

What's in the box ?

Check that you have all these parts

- 1 SureSense Microchip Reader
- 2 USB cable (for firmware updates only)
- 3 Temperature Sensing Microchip information leaflet

Batteries not included

Contents

Important information	3
Technical specifications	3
Batteries	4
How to take a reading	4
Changing the temperature unit between °C & °F	4
Display	5
Firmware updates	5
Troubleshooting	5
Warranty and disclaimer	6

Important information

PLEASE READ THE FOLLOWING INFORMATION CAREFULLY AND RETAIN THIS MANUAL FOR FUTURE REFERENCE.

- You can find the **product code** and **serial number** of the Microchip Reader on the inside of the battery cover.
- Please make sure you have all the parts listed on **page 1**.
- **To register this product's warranty visit: sureflap.com/suresense-warranty**
- Only use the USB cable provided and **do not insert any foreign objects into the product**.
- This product is **not waterproof**.
- The reader **requires two alkaline AA batteries**, which should be inserted and replaced with caution. **Do not mix different types** of batteries or **new and used** batteries. If the unit is going to be **unused** for a long period, **please remove the batteries**.

Safety warnings

- **This product uses RFID (Radio Frequency Identification) and therefore may cause problems with sensitive electronic medical devices such as pacemakers and monitors within a 1 metre radius.**
- Do not use this product if parts are missing, damaged or worn.
- **Keep small parts out of reach of children and animals.** Children should not be permitted to play with the SureSense Microchip Reader. It is not a toy and should not be treated as such. However, the Microchip Reader can be used by children aged 8 years and above and also by persons with reduced physical, sensory or mental capabilities as long as they are supervised.

Technical specifications

Size:

136mm (L) x 63mm (W) x 29mm (H) (5 3/8" x 2 1/2" x 1 1/8")

Battery life:

6 months (Typical usage at 3 reads per day, with good quality alkaline AA batteries)

Compatible microchips:

ISO FDX-B (15-digit)

Readable temperature range:

+33°C to +43°C (+91.4°F to +109°F)

If the Microchip Temperature doesn't fall within this range an out of range symbol will be displayed, see page 5.

Maximum reading distance:

Up to 95mm (3 3/4") away from the microchip, depending on microchip type and orientation.

Storage temperature:

-10 °C to +70 °C (+14°F to +158°F)

Operating temperature:

-5 °C to +55 °C (+23°F to +131°F)

Working frequency:

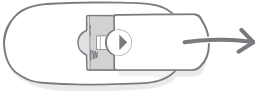
134 kHz (RFID)

AA x2

Batteries


To insert batteries please follow the steps below.

1

Remove the battery cover by pressing the arrow and sliding it away from the centre.

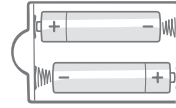2

Insert the batteries making sure the + and - symbols are the correct way round.


This icon will be displayed when the batteries begin to run low. Always use **2 high quality alkaline AA batteries**.

How to take a reading

To read an animal's microchip, please follow the steps detailed below.

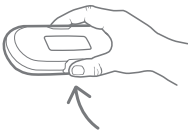1

Press the button on the reader to turn it on. (It will automatically turn off when not in use)

2

Press the button again and move the reader around the animal, following one of the **Motion Patterns** detailed below to locate the microchip.

3

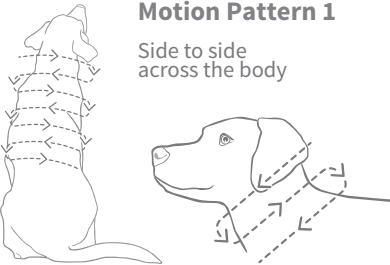
When the reader detects the microchip it will display its number and, if applicable, Microchip Temperature.


To locate the microchip, hold the reader close to the animal's neck, then move it in either one of the suggested Motion Patterns pictured below.

Motion Pattern 1

Side to side across the body


**Motion Pattern 2**

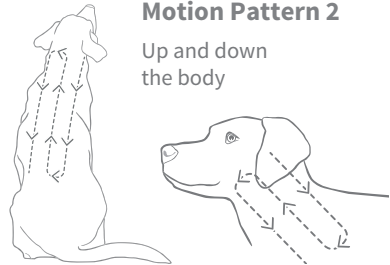Up and down the body

Changing the temperature unit between °C & °F

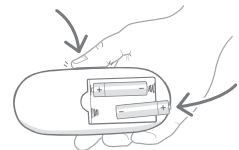To change the unit of temperature for the reader, first remove the batteries. Then press and hold the reader's button whilst re-inserting the batteries. The Microchip Temperature will now be displayed in Fahrenheit. To revert back to Celsius, repeat the process.

Display

The standard display for the reader is:

Microchip ISO FDX-B 15-digit number

Battery level indicator

Microchip Temperature

(Only for temperature sensing microchips)

One of the icons below will replace the temperature reading if it is **out of range**:

Below 33 °C
(91.4 °F)

Above 43 °C
(109 °F)

Displayed when the reader is **searching** for the animal's microchip.

Displayed when the **batteries are completely flat**. To insert new batteries see page 4.

Firmware updates

The USB cable provided is for firmware updates only and cannot be used for charging or powering. For firmware updates and further information go to: sureflap.com/suresense-support

Troubleshooting

The reader won't turn on: Make sure the batteries are inserted in the correct orientation. If this doesn't solve the problem, try changing the batteries, see page 4 for details.

No temperature is displayed: If the reader only displays the microchip number, this may indicate that the animal does not have a temperature sensing chip.

The reader will not detect the animal's microchip: This could be as a result of either of the following:

- The animal's microchip was **out of reading range whilst scanning**, meaning it wasn't detected in the area scanned with the reader. To improve your chances of locating an animal's microchip, try either of the scanning motion patterns on page 4.
- The animal doesn't have a compatible microchip, or has no microchip at all. **The reader only works for animals with ISO FDX-B 15-digit microchips.**
- Some animals may have more than one microchip implant, in this case microchips close to each other could affect the reader performance. To improve your chances of reading one of the microchips, try either of the scanning motion patterns on page 4.
- **Metal may interfere with the reader**, try taking a reading away from large metal objects such as a metal table.

Still in need of assistance?

You can find additional help or contact information at : sureflap.com/suresense-support

Warranty: The SureSense Microchip Reader carries a 2-year warranty from the date of purchase, subject to proof of purchase date. The warranty is restricted to any fault caused by defective materials, components or manufacture. This warranty does not apply to products whose defect has been caused by normal wear and tear, misuse, neglect or intentional damage.

In the event of a part failure due to faulty parts or workmanship, the part will be replaced free of charge during the warranty period only. At the manufacturer's discretion a replacement product may be provided free of charge in the case of a more serious malfunction. Your statutory rights are not affected.

To register your warranty visit: sureflap.com/suresense-warranty 

Disclaimer: The SureSense Microchip Reader, including its use to measure Microchip Temperature is not intended to replace traditional veterinary advice. If you think you need veterinary advice you should take your pet to a veterinary practice. We expressly disclaim any liability for any loss, damage or costs that you may incur as a result of using the product as a substitute for professional veterinary advice. Furthermore we do not accept any liability for any veterinary fees that you may incur as a result of your use of the product.

Body temperature is not enough to determine disease alone. Microchip Temperature is not a replacement for a rectal temperature measurement by a veterinarian, who is the only person qualified to determine your pet's health condition. Microchip Temperature and rectal temperature, while correlated, are not equivalent and should not be expected to show identical readings.

Disposal of Products and used Batteries: This icon will indicate products that will be subject to the following legislation:

The Waste Electrical and Electronic Equipment Directive 2012/19/EU (WEEE Directive) & The Battery Directive 2006/66/EC and Amendment 2013/56/EU places an obligation on households to dispose of the waste batteries and products that have reached the end of their life, in an environmentally responsible manner as this will reduce the impact the waste will have on the environment, therefore these cannot be placed in household waste bins and should be recycled at an appropriate recycling facility.

FCC compliance: This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

- (1) this device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This device has been designed and complies with the safety requirements for portable (<20cm) RF exposure in accordance with FCC rule part §2.1093 and KDB 447498 D01.

IC compliance: This device complies with Industry Canada license-exempt RSS standards. Operation is subject to the following two conditions:

- (1) This device may not cause interference, and
- (2) This device must accept any interference, including interference that may cause undesired operation of the device.

This device complies with the safety requirements for RF exposure in accordance with RSS-102 Issue 5 for portable use conditions.

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Copyright © SureFlap Ltd, 7 The Irwin Centre,